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Instructions for colouring of a posterior tooth

Step	Colour Liquid	Colour application	buccal	occlusal
1	A3 Prettau® Aquarell Colour Liquid	1x 2x 3x		
2	A3 Prettau® Aquarell Colour Liquid	1x 2x		
3	A3 Prettau® Aquarell Colour Liquid	1x		
4	Cervical A Aquarell Colour Liquid for Prettau	1x 2x 3x buccal occlusal		
5	Incisal blue Aquarell Colour Liquid for Prettau	1x - 2x		
6	Incisal grey Aquarell Colour Liquid for Prettau	1x		
7	Incisal violet Aquarell Colour Liquid for Prettau	1x		
8	Orange 1 Aquarell Colour Liquid for Prettau	1x		
	Brown 2 Aquarell Colour Liquid for Prettau	1x		

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Instructions for colouring of a front tooth

Step	Colour Liquid	Colour application	labial	palatal
1	A3 Prettau® Aquarell Colour Liquid	1x 2x 3x		
2	A3 Prettau® Aquarell Colour Liquid	1x 2x		
3	A3 Prettau® Aquarell Colour Liquid	1x		
4	Cervical A Aquarell Colour Liquid for Prettau	1x		
5	Incisal blue Aquarell Colour Liquid for Prettau	1x - 2x		
6	Incisal grey Aquarell Colour Liquid for Prettau	1x		
7	Incisal violet Aquarell Colour Liquid for Prettau	1x		
8	Orange 1 Aquarell Colour Liquid for Prettau	1x		
	Brown 2 Aquarell Colour Liquid for Prettau	1x		